


Practicon® Dr. Schwartz's Safety Tie™ Instrument Leashes

Instructions

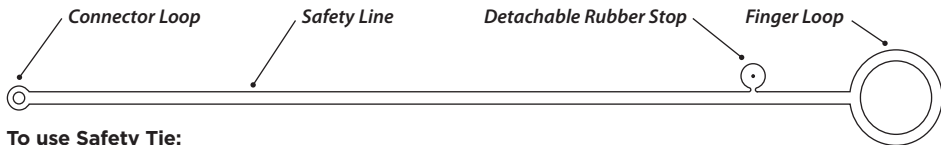


To use Safety Tie:

1. Thread tip of driver through connector loop.
2. With tip still through connector loop, thread tip of driver through detachable rubber stop so that connector loop is retained by rubber stop. The rubber stop has a small hole pre-punched in the center to assist with tip insertion.
3. Tear rubber stop off safety line with a firm pull.
4. Place finger through loop for security, if desired.
5. Remove and dispose of Safety Tie after procedure.

Practicon, Inc. • 1-800-959-9505

**Keep these instructions
for future reference.**

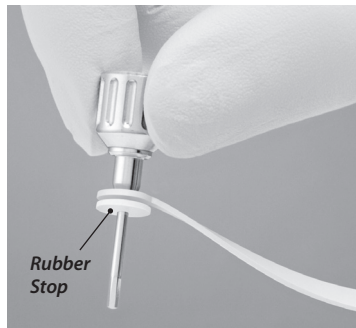

Correct position of rubber stop

**5mm Inner
Diameter**

Dr. Schwartz's Cap Trap™ Instructions for Use

**6mm Inner
Diameter**

1. Select a Cap Trap with inner diameter slightly smaller than healing abutment outer diameter, and press Cap Trap over abutment. Cap Trap may be cut to length for use with shorter hex drivers.
2. Insert hex driver through Cap Trap and loosen abutment. Remove hex driver.
3. With Cap Trap still gripping abutment, rotate and remove abutment using Cap Trap. Abutment may be replaced using Cap Trap as a handle. Once engaged, remove Cap Trap and retighten with hex driver.
4. Dispose of Cap Trap after use.

**3mm Inner
Diameter**



Do not reuse. This is
a single use device.



Not made with
natural rubber latex.

**4mm Inner
Diameter**

Practicon, Inc. • 1-800-959-9505