

Practicon® DebrisSafe™ Workstation

Instructions for Use

Thank you for purchasing the **DebrisSafe Workstation**. With proper care, it will provide long lasting protection for staff and work spaces from airborne particles associated with grinding or sandblasting. A replaceable inner shield guards the acrylic window from pits and scratches over time.

For Best Results:

1. To empty the plastic collection tray, simply lift off the clear acrylic working chamber. The tray is made of high-density polypropylene (HDPE) and may be wiped with any cleaner/disinfectant.
2. DO NOT clean or disinfect the clear acrylic chamber with alcohol- or bleach-based cleaners. Wash acrylic chamber with soap and water only and allow to air dry.
3. Inner shield may be washed with soap and water, if necessary. Replace the inner shield when it is scratched or cloudy. For best results, do not use DebrisSafe without inner protective shield in place.

To Replace Inner Shield:

1. Turn acrylic working chamber upside down.
2. Catch the front edge of the inner shield and pull it away from the edge stops (see Figure 1). Lift used shield out.
3. Remove protective film from both sides of new shield. Reverse inner shield removal procedure to install new shield.



Figure 1

Manufactured for

 **PRACTICON**

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834

Made in USA • IFU 7001741_1

70-01742 DebrisSafe Inner Shields.
3/pack.