



# Practicon® Disc Safe™ Polishing Disc Holders

## 70-01753

### Contents:

- (2) Stainless Steel Organizers
- (5) Interchangeable Silicone Grip Bands

- Grips Shofu® Super-Snap Discs and other similar finishing and polishing discs for quick, one-handed loading and unloading.
- Saves time and motion spent switching discs mid-procedure.
- Organizer and grip bands are fully autoclavable.

---

## Instructions (photos on other side)

1. Select silicone grip band color of your choice, and thread band into T-slots on both sides of organizer as shown (Figure 1).
2. Preload organizer with up to four discs by sliding silicone disc shanks into slots until secure (Figure 2).
3. Pick up a disc by firmly pressing mandrel into disc shank and sliding disc out of slot (Figure 3).
4. Unload a used disc by sliding shank back into slot and pulling mandrel away from disc. Repeat for other grits/sizes. (Figure 4).

Manufactured for



# PRACTICON

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834


Made in USA • Mixed Case 7001753\_1




723896029060




**Figure 1.** Thread a silicone grip band into T-slots on both ends of organizer.

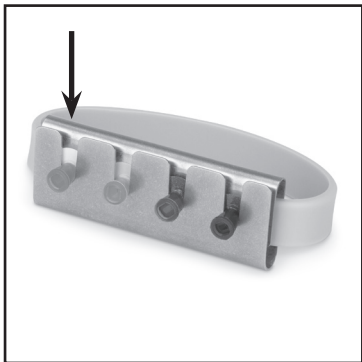

**Figure 2.** Preload organizer with up to four discs by sliding silicone disc shanks into slots until secure.

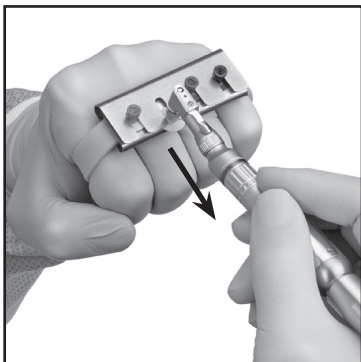

**Figure 3.** Pick up a disc by firmly pressing mandrel into disc shank and sliding disc out of slot.

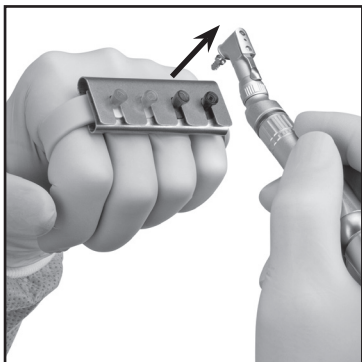

**Figure 4.** Unload disc by sliding shank back into slot and pulling mandrel away from disc.