



Practicon® PRACTIview™ 2 Operator Mirror

Instructions for Use

Thank you for purchasing the **PRACTIview 2 Operator Mirror**. Designed to fit all styles of operatory lights and support arms, the lightweight PRACTIview 2 installs easily without tools, is easy to adjust for every patient situation, and will not affect normal light operation.

Installation:

1. Select an installation point on your operatory light or other support arm.
2. Wind the end of the flexible mirror support tightly around the light support arm two or more times (Figures 1 and 2).
3. Bend the end of the flexible support tightly over itself 180° as shown (Figure 3).
4. Position the mirror over the patient's face as needed.



Figure 1



Figure 2



Figure 3



PRACTICON

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834

Made in USA • IFU 701725_1