

Contours Soft Disposable Prophylaxis Angles

Instructions for use:

Place disposable prophylaxis angle (DPA) onto nose cone of a standard low speed or doriot-style handpiece. If handpiece is designed with a friction grip (no collet), align notch in DPA body with locator pin or tab on low speed handpiece and push DPA firmly into handpiece until fully seated. If handpiece uses collet retention, insert DPA into handpiece until it is fully seated. Turn collet to lock DPA in place, then check for proper grip by gently pulling on DPA. DPA should remain fully seated and locked in place.

Caution:

- This is a disposable product. Dispose of properly after one use. DO NOT attempt to sterilize or reuse.
- 1,500 to 3,000 RPM operation recommended. DO NOT operate above 3,000 RPM.



Do not reuse. This is a single use device.



Store in a dry place protected from light and heat.



Not made with natural rubber latex.



Read instructions before use.




For professional use only.

US Federal law restricts this device to sale or use on the order of a licensed practitioner.



Non-sterile

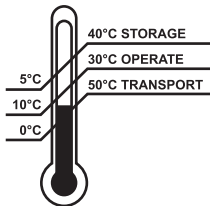

**Practicon, Inc.
1112 Sugg Pkwy
Greenville, NC 27834 USA**