

Instructions for Use

Thank you for your SDFlosser® purchase. The SDFlosser was developed by a dentist to speed and simplify interproximal application of silver diamine fluoride (SDF).

1. Connect an SDFlosser to a standard saliva ejector and turn on suction.
2. Dry tooth surface(s) to be treated and isolate using absorbent triangles or cotton rolls, if necessary.
3. Place SDFlosser into patient's mouth and insert floss between teeth.
4. Dip a standard micro applicator into SDF and saturate floss (Figure 1).
5. Remove SDFlosser after one minute, avoiding the soft tissues upon exit.
6. If treating another interproximal area on same patient, pull to remove SDFlosser tip and replace with a new tip (Figure 2). Repeat as above.
7. For single patient use only. Non-autoclavable. Discard all components after use.

Keep these instructions for future reference.



Figure 1



Figure 2