

# PRACTIshield™

## Directions for use. Use only as directed.

1. Attach capped needle to syringe. Before uncapping needle, insert needle sheath through hole in PRACTIshield until sheath collar is firmly seated against shield surface.
2. While holding sheath, remove syringe and set shield and sheath onto a flat surface.
3. Using PRACTIshield as a prop, **resheath needle with ONE HAND** by inserting needle into opening of sheath. If necessary, tip syringe upwards and press down to secure needle into the sheath.
4. Dispose of needle, sheath, and shield properly.



PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • [practicon.com](http://practicon.com)