



# Practicon® Cool Cassette™ 2 Instrument Tray

The Cool Cassette Instrument Tray provides many unique features and advantages over a traditional metal tray, including faster heat dissipation and safer, quieter, soft-touch handling. 100% silicone construction with no moving parts ensures a long service life.

After opening, the lid may be placed underneath the base to save working space. To close, press the lid onto the base, being certain to fully engage all side lock buttons.

Because of the silicone rubber construction, the interior instrument saddles can be cut with scissors to accommodate large-handled instruments, if desired. Use caution while cutting.

**See instructions below.**



Read instructions  
before use.



Not made with  
natural rubber latex.

Cool Cassette 2 Instrument Tray withstands most cleaning solutions and is heat sterilizable to 375°F (190°C). The longevity of the tray depends on actual use and is normally determined by visual degradation. Discontinue the use of the tray immediately if outer walls become cut or damaged. DO NOT use Cool Cassette to hold instruments longer than 7½" in length.

For sterilization processing, follow the instructions from the manufacturer of your instruments and cleaning and sterilization equipment. Cool Cassettes make NO claim and are NOT designed to maintain the sterility of contents inside after sterilization without prior enclosure in sterilization bags or wraps designed and approved for that purpose.

If Cool Cassettes are submerged in a liquid cleaning solution prior to sterilization (in presoak or ultrasonic cleaner), use any solution other than those containing bleach (sodium hypochlorite) or sodium bicarbonate. Over long periods of time, bleach can weaken and discolor silicone materials. Solutions containing sodium bicarbonate may evaporate and leave a salty deposit or film on the surface of Cool Cassettes. This film is harmless and can be removed with soap and water if necessary.

Practicon, Inc. • 1-800-959-9505

**Keep these instructions for future reference.**

Manufactured for

US Pats. 9,610,126 D903,901

 **PRACTICON**

PRACTICAL INNOVATIONS FOR DENTISTRY

1-800-959-9505 • Fax 1-800-278-0885

practicon.com • Greenville, NC 27834

Made in USA • Insert 7141910-14\_3