


Practicon[®] Dr. Schwartz's Safety Tie[™] Instrument Leashes

Instructions

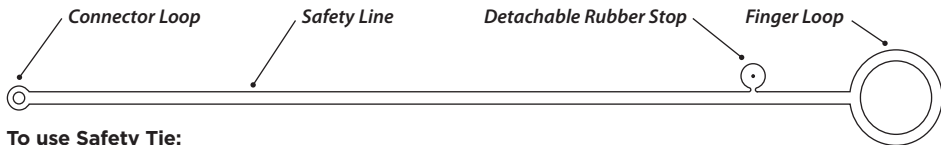


To use Safety Tie:

1. Thread tip of driver through connector loop.
2. With tip still through connector loop, thread tip of driver through detachable rubber stop so that connector loop is retained by rubber stop. The rubber stop has a small hole pre-punched in the center to assist with tip insertion.
3. Tear rubber stop off safety line with a firm pull.
4. Place finger through loop for security, if desired.
5. Remove and dispose of Safety Tie after procedure.

Practicon, Inc. • 1-800-959-9505

**Keep these instructions
for future reference.**

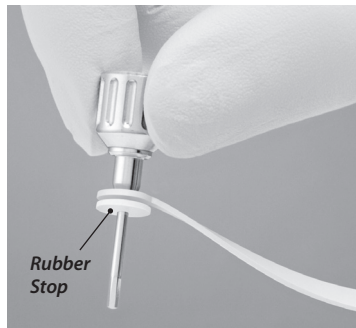

Correct position of rubber stop