

# Practicon® **Lock-On™** **HVE Hose Adapter**

## Instructions for Use

Thank you for your purchase of the **Lock-On HVE Hose Adapter**, designed to connect any low volume suction device to any HVE valve by way of a 60" long, lightweight, vinyl hose. Enjoy reduced hand fatigue and better maneuverability.

### For proper use:

1. Firmly insert U-shaped adapter into HVE valve. Leave HVE valve in dental unit. Use valve lever to turn suction ON/OFF.
2. Insert low volume aspirator into Lock-On silicone tip.  
**Important: To better support the weight of the hose, insert the aspirator until it stops (approx. 1") as shown at right.**
3. Use the built-in clip on U-shaped adapter to hang free end of hose when not in use.
4. To clean, simply flush along with other evacuator lines. U-shaped adapter and hose may be wiped with surface disinfectant **except products containing phenol or hydrogen peroxide. Do not autoclave.**

**Correct aspirator insertion**



**Incorrect aspirator insertion**

